

The AcuTip 500's specially designed tip (6.35 mm in diameter) allows treatment only where needed for safe focal treatment of pigmented and vascular lesions. AcuTip 500's emission band (500-635 nm) is ideal for aggressive absorption of hemoglobin and melanin. Used alone, or in conjunction with a large area broadband treatments, AcuTip 500 provides a solution to maximize single treatment results.



### Practice benefits:

- High efficacy and safety
- No downtime
- Safe on skin types I - IV
- Enables pinpoint treatment
- No disposables

## Lentigo

Skin type 4, Japanese, Mid-40's, Female

## Treatment Protocol

Fluence: 12 J/cm<sup>2</sup>

# of treatments: 1 Tx



Before




8 weeks after 1 Tx

### Treatment progress: Video microscope x50 (polarized)



Before

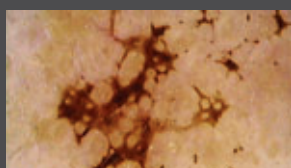

Day 1



Day 7



Day 21

Photos courtesy of Kei Negishi, M.D. Tokyo Women's Medical University  
Aoyama Institute of Women's Medicine, Tokyo, Japan