

Using a novel wavelength, YSGG (2790 nm), Pearl offers full epidermal turnover. Its minimally invasive technology renews the skin's surface by treating fine lines, texture, age spots and sun damage with fewer treatments and less overall downtime than fractional non-ablative lasers.

Practice benefits:

- No disposables
- Minimal social downtime (3 - 5 days)
- 1 - 2 treatments
- Safe on skin types I - IV
- High patient satisfaction



Photoaging

Skin type II, Caucasian, Female

Treatment Protocol

Fluence: 3.5 J/cm²
Pulse width: 0.4 ms
Overlap: 20 %
of treatments: 2 Tx
of passes: 2 passes (add 2nd pass on periorbital, perioral & seborrheic keratoses)



Before



5 Weeks After 2 Tx

Photos courtesy of Debra Jaliman, MD