

Pearl Fractional creates deep columns of ablation with a controlled zone of thermal damage. Controlling thermal damage maximizes the benefits of thermal stimulation and hemostasis without the added pigment complications associated with excessive thermal damage. For these reasons, Pearl Fractional is a superior fractional ablative solution to both CO₂ and Er:YAG.



Practice benefits:

- Minimal social downtime (5 - 7 days)
- Low risk, high efficacy
- High patient satisfaction
- 1 - 2 treatments
- Safe on skin types I - IV

Skin Texture


Skin type II, Caucasian, Male

Treatment Protocol

Entire face (Includes eyelids)
Fluence: 160 J/cm²
Density level: Density 3
1 pass (2 passes on cheeks)

Eyelids

Fluence: 120 J/cm²
Density level: Density 2 - first pass
Density 1 - second pass

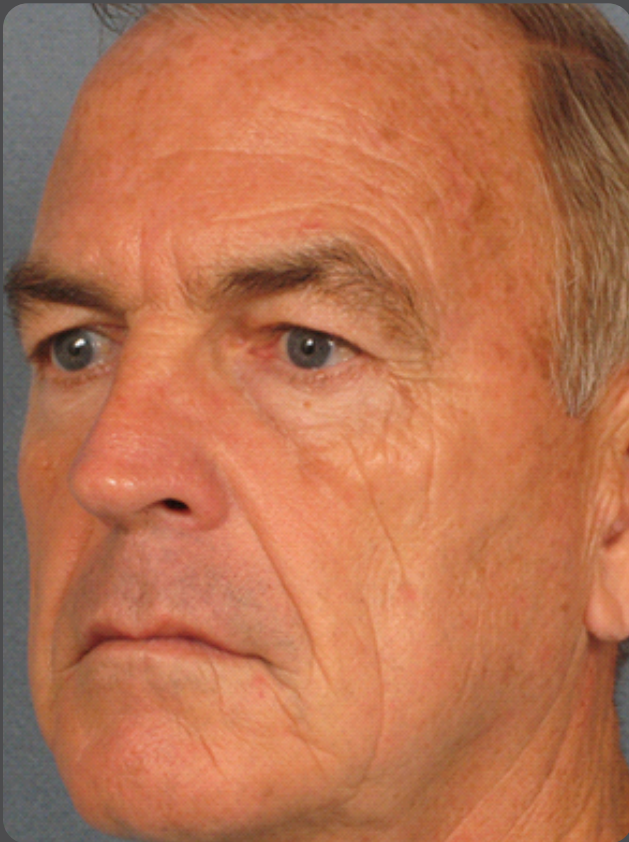

Before



6 weeks after 1 Tx

Photos courtesy of Janet Turkle, MD.